



» Novaloc «

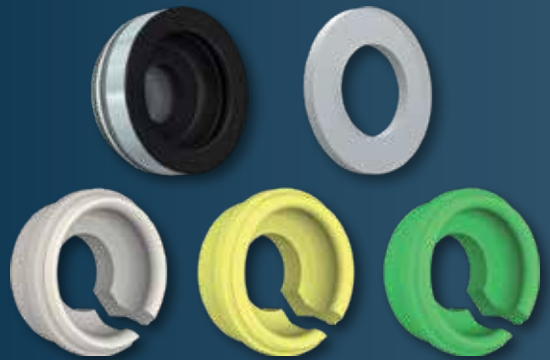

10 Reasons To Choose Novaloc™

① Matrix housing made of titanium or PEEK

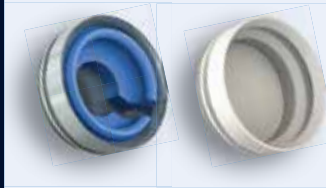

Matrix housing in beige PEEK; the solution when there is very little space or when not using metal is a prerequisite.

② Stress-free mounting and removing of retention inserts



1. Mount retention inserts



2. Insert into the matrix housing



3. Clearly audible "click"



1. Hold demounting tool vertically over the retention insert

2. Set up with slight pressure

3. Remove vertically upwards



③ The ingenious Novaloc™ model analogue reposition aid



1. Mount model analogue



2. Insert model analogue



Absolutely safe handling



1. Use the stainless steel finish

2. Insert at an angle into the bore of the mounting insert

3. By tipping easily removed from the matrix housing



④ The clever demounting tool for mounting inserts

⑤ The user-friendly **Novaloc™** equipment box



- No gaps in the inventory
- Ensures overview and order
- No mixing when the cover is closed
- Ordering number legible in the cover
- Three special instruments for all sorts of manipulations

⑥ The “5-seconds” matrix housing extractor



1. Heat extraction head


2. Heat matrix housing briefly

3. Apply leverage to remove matrix housing



Special design:
No more hot finger!

⑦ **Novaloc™** only requires 4 replacement males

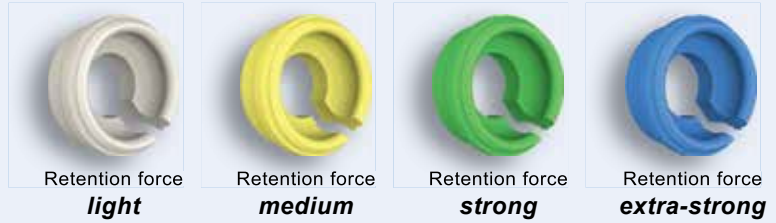

Easy to understand colour code according to retention force

⑧ New forming/fixing matrix for a safe and accurate positioning in the mouth



Now clearly smaller and more precise than those forming matrices commercially available until now

Multivalent can also be used as a fixing matrix for templates, bite registers and rails



⑨ Processing spacer now with blocking characteristics



A 5° inclination for the shaping of a self-retaining box for matrix attachment














For additional mechanical blocking

⑩ Clearly less wear and tear as well as servicing expense at the retention inserts

- **Novaloc™** retention inserts need no central retention element (considerable advantage with regard to damage)
- PEEK has better physical properties and is considerably more hygienic than Nylon



Conventional damage to Nylon inserts commercial available until now

Picture	Part no.	Part description	Specifications	Amount per package	Price \$
	2010.101	Novaloc™ Equipment Box	Incl. 3 tools (without load)	1 pc	\$ 285.00
	2010.601	Novaloc™ Processing package titanium	· Titanium matrix housing incl. mounting insert (1 pc) · Retention insert white (1 pc) · Retention insert yellow (1 pc) · Retention insert green (1 pc) · Mounting collar, silicone (1 pc)	2 pcs	\$ 95.00
	2010.611	Novaloc™ Processing package PEEK	· PEEK matrix housing incl. mounting insert (1 pc) · Retention insert white (1 pc) · Retention insert yellow (1 pc) · Retention insert green (1 pc) · Mounting collar, silicone (1 pc)	2 pcs	\$ 95.00
	2010.701	Novaloc™ Matrix housing, titanium (incl. black mounting insert)	Matrix housing: titanium Mounting insert: PEEK	4 pcs	\$ 95.00
	2010.702	Novaloc™ Matrix housing, PEEK (incl. black mounting insert)	Matrix housing: PEEK Mounting insert: PEEK	4 pcs	\$ 95.00
	<u>special accessory</u> 2010.703	Novaloc™ Matrix housing, titanium with attachment option (incl. black mounting insert)	Matrix housing: titanium Mounting insert: PEEK	4 pcs	\$ 95.00
	<u>special accessory</u> 2010.710	Novaloc™ Retention insert red	PEEK Retention force extra-light 0.5165	4 pcs	\$ 57.00
	2010.711	Novaloc™ Retention insert white	PEEK Retention force light 1.5	4 pcs	\$ 57.00
	2010.712	Novaloc™ Retention insert yellow	PEEK Retention force medium 2.5	4 pcs	\$ 57.00
	2010.713	Novaloc™ Retention insert green	PEEK Retention force strong 3.5	4 pcs	\$ 57.00
	2010.714	Novaloc™ Retention insert blue	PEEK Retention force extra-strong 4.5	4 pcs	\$ 57.00
	<u>special accessory</u> 2010.715	Novaloc™ Retention insert black	PEEK Retention force ultra-strong 5.5	4 pcs	\$ 57.00
	56	Analog	Stainless Steel	1 pc 10 Pack	\$ 22.00 \$ 190.00
	2010.722	Novaloc™ Forming / fixing matrix, red	PEEK	4 pcs	\$ 85.00
	2010.723	Novaloc™ Processing spacer, white	POM C	4 pcs	\$ 45.00
	2010.724	Novaloc™ Mounting collar, silicone	Silicone	10 pcs	\$ 32.00
	2010.725	Novaloc™ Mounting insert, black	PEEK	4 pcs	\$ 52.00
	2010.731	Novaloc™ Demounting tool for mounting inserts + model analogue reposition aid (blue)	Aluminium, steel	1 pc	\$ 75.00
	2010.741	Novaloc™ Mounting and demounting tool for retention inserts (brown)	Aluminium, steel	1 pc	\$ 75.00
	2010.751	Novaloc™ Matrix housing extractor (grey)	Aluminium, steel	1 pc	\$ 75.00



Novaloc[®]
swiss made

TIRED OF WORN ABUTMENTS?

Novaloc[®] abutments set new benchmarks in wear resistance.



LIFETIME COATING (LTC) UTILIZES
A DIAMOND LIKE CARBON
COATING

10 YEARS WARRANTY

TITANIUM NITRIDE
SURFACE

1 YEAR WARRANTY

NEW STANDARD



Simulation of 3.35 Million snap cycles on the Novaloc matrix system

PREVIOUS STANDARD



Simulation of 10,000 snap cycles on the Novaloc matrix system

Product Image	Part no.	Part description	Specifications
	048.189V2	Locator male processing package, 2-pack	denture cap with black processing male, block-out spacer, clear replacement male, pink replacement male, blue replacement male, Ti/Nylon
	048.192V4	Locator replacement male, blue, extra-light retention, 4-pack	0-10 degrees, 1.5 lbs, 0.68 kg, height 1.7mm, Nylon
	048.191V4	Locator replacement male, pink, medium retention, 4-pack	0-10 degrees, 3.0 lbs, 1.36 kg, height 1.7mm, Nylon
	048.190V4	Locator replacement male, clear, strong retention, 4-pack	0-10 degrees, 5.0 lbs, 2.27 kg, height 1.7mm, Nylon
	048.182V2	Locator male processing package, extended range, 2-pack	denture cap with black processing male, block-out spacer, green replacement male, orange replacement male, red replacement male, Ti/Nylon, 10-20 degrees
	048.194V4	Locator replacement male, red, extra-light retention, extended range, 4-pack	10-20 degrees, 1.0 lbs, 0.45 kg, height 1.7mm, Nylon
	048.188V4	Locator replacement male, orange, light retention, extended range, 4-pack	10-20 degrees, 2.0 lbs, 0.91 kg, height 1.7mm, Nylon
	048.193V4	Locator replacement male, green, strong retention, extended range, 4-pack	10-20 degrees, 4.0 lbs, 1.82 kg, height 1.7mm, Nylon
	048.181V4	Locator replacement male, grey, zero retention, extended range, 4-pack	10-20 degrees, 0.0 lbs, 0.0 kg, height 1.7mm, Nylon
	048.8510	Locator Denture cap with processing male, 4-pack	titanium
	048.195V4	Locator black processing male, 4-pack	nylon, height 1.9mm
	048.196V20	Locator block-out spacer, 10-pack	thickness 0.4mm, silicone rubber
	048.197V4	Locator impression coping, 4-pack	titanium, height 4.0mm, Al housing with LDPE insert
	56	Lab analog	titanium
	048.218V4	Locator processing spacer, 4-pack	POM
	046.415	Locator Core Tool	titanium

**Office hours: Monday - Thursday 9am - 6pm EST
Friday 9am - 5:30pm EST**

Next day delivery on orders placed before 5:30pm EST

Free Shipping on orders over \$500.00 (before taxes)

Abutment Direct Inc.

16-250 Shields Court, Markham, ON Canada L3R 9W7

Phone: (905) 604-0465 Fax: (905) 604-0900 Toll Free: 1-855-604-0465

info@abutmentdirect.com www.abutmentdirect.com